

AXO RCS

ROOT CANAL OBTURATION CEMENT



Product Description :

Axo RCS is a radiopaque, non-resorbable, Zinc Oxide Eugenol based root canal sealer. Its composition is well tolerated by the tissues and provides anti inflammatory action

Composition :

Composed of Powder & Liquid

Powder :Zinc Oxide, Barium Sulphate, Thymol Iodide, Magnesium Stearate.

Liquid : Eugenol & excipients.

Indications :

1.Root canal filling sealing in association with a gutta percha point.

Instructions for Use :

Mix powder and liquid in for around 50-60 seconds, one scoop of powder and four drops of liquid to obtain a thick mix, which can easily be introduced into the canal with a suitable root canal gutta percha point or in conjunction with a silver point. setting time is 45-60 minutes from start of mix

Contraindication :

It should not be used if the patient is known to be allergic to any of the ingredient used.

Caution :

- 1) In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water. To avoid contact, a rubber dam and/or local butter can be used to isolate the operation field from the skin or oral tissue.
- 2) In case of contact with eyes flush immediately with water and seek medical attention.

Storage Conditions :

Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf life :

3 years from the date of manufacturing.

Presentation :

Powder 15g, Liquid 10ml, 1x Spoon, 1 x Mixing Pad, 1 x Instruction Manual

Disposal conditions :

To be disposed off based on the prevailing rules of the country in

Manufactured by
Dentaxon
3/222C, 3/222D
Malappuram, Kerala -679328
info@dentaxon.com
www.dentaxon.com

| | |
|------------|--|
| Revision : | |
| Date : | |

Symbol for "HARMFUL IRRITANT"

Symbol for "CORROSIVE"

Symbol for "BATCH CODE" Symbol for "USE BY" Symbol for "NON STERILE" Symbol for "KEEP DRY"

Symbol for "CAUTION" Symbol for "CONSULT INSTRUCTION FOR USE" Symbol for "CATALOGUE NUMBER"

SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"

Symbol for "KEEP OUT OF THE REACH OF CHILDREN" Symbol for "MANUFACTURER NAME & ADDRESS"

Symbol for "TEMPERATURE LIMITATION" Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.