

AXO CEM-LC



Light Curing Glass Ionomer Composite Cement

Product Description : Axo Cem-LC single component, light cured glass ionomer cement for lining, extended fissure sealing and smaller lesions.

Composition : Urethane Dimethacrylate resin, Bisphenol A Glycidyle Methacrylate, Barium Sulphate, FluoroAluminoSilicate Glass Powder, Silanated Barium Glass Powder, Amorphous Fumed Silica, Photoinitiators and Stabilizers

Indications : ♦ Lining under all kinds of filling materials ♦ Extended fissure sealing ♦ Restoration of smaller lesions.

Instructions for use : Thoroughly clean and air dry the cavity. Apply material to the cavity directly with the help of applicator tip. Do not exceed the thickness of 1mm. Light cure for 30- 40 sec using standard curing light and proceed with standard restoration technique.

Warning : Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

Contraindications : Fusion I-Seal is contraindicated for use in patients with known allergies to methacrylate or any other if its components.

Storage Conditions : Store in a cool and dark place between 10°C to 24°C. Keep away from high temperature as direct sunlight. Close the syringe carefully after use. Keep away from moisture. Shelf Life not affected after opening if stores in a cool dry place. If not to be used for a longer duration, keep them in refrigerator. Allow material to reach room temperature prior to use.


Shelf Life :

3 years from the date of manufacturing.

Presentation :

1 x 2g Syringe, 3 x Applicator Tips

Disposal conditions : To be disposed off based on the prevailing rules of the country in which it is used.

 Manufactured by
Dentaxon
3/222C,3/222D
Malappuram, Kerala -679328
info@dentaxon.com
www.dentaxon.com

Revision :	
Date :	

 LOT Symbol for "BATCH CODE"	 Symbol for "USE BY"	 Symbol for "NON STERILE"	 Symbol for "KEEP DRY"
 Symbol for "CAUTION"	 Symbol for "CONSULT INSTRUCTION FOR USE"	 Symbol for "CATALOGUE NUMBER"	
 SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE	 Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"		
 Symbol for "KEEP OUT OF THE REACH OF CHILDREN"	 Symbol for "MANUFACTURER NAME & ADDRESS"		
 Symbol for "TEMPERATURE LIMITATION"	 Symbol for "KEEP AWAY FROM SUNLIGHT"		

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.