

# AXO ETCH

## 37% Phosphoric Acid Etchant Gel

**Product Description :** Axo Etch is a blue tinted viscous gel of 37% Phosphoric acid for etching tooth enamel and dentin conditioning or composite surfaces prior to the bonding of any composites, sealants or bonding adhesives.

**Composition :** Phosphoric Acid, Methylene Blue Indicator, Glycerine, Ethanol, Purified Water and Amorphous Silica.

**Indications :** Used for conditioning both dental enamel and dentine surfaces prior to the placement of an adhesive restoration.

**Instructions for Use :** Clean the area to be etched, rinse and dry thoroughly. Place the tip against the surface to be etched and slowly dispense the gel on the surface to lay a fine line or paint a thin layer to the areas to be etched. Allow the gel to stay for 10-15 seconds and rinse thoroughly with water spray. Dry etched tooth surface with clean, dry air.

**Precautions :** The product contains phosphoric acid. It can burn and cause irritation to the tissues and therefore confine the use to areas to be etched. Fix the applicator tip to the syringe and check the flow of gel before applying on the etching surface. Avoid contact with eyes and prolonged contact with skin.

**Warning :** Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

### Storage Conditions :

Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

**Shelf Life :** 3 years from the date of manufacturing.

### Presentation :

1 x 5ml Syringe, 3 Applicator Tips.

### Disposal conditions :


To be disposed off based on the prevailing rules of the country in which it is used.



Manufactured by  
Dentaxon  
3/222C,3/222D  
Malappuram, Kerala -679328  
info@dentaxon.com  
www.dentaxon.com

CE  
0123

Revision :	
Date :	

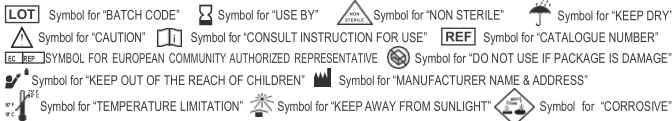

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.