

Axo Flow

Light Cured Nano Hybrid Flowable Composite



Product Description : Axo Flow is an excellent flowable composite that delivers high strength & wear resistance. It is Characterized by optimum surface affinity, penetrates into most difficult area. It combines esthetics as well as flow on demand handling for use.

Composition : Bisphenol A Glycidyl Methacrylate, Ethoxlated Bisphenol A Dimethacrylate, Urethane Dimethacrylate, Triethylene Glycol Dimethacrylate, Silanated Barium Glass Powder, Amorphous Fumed Silica, Total inorganic filler 70%.

Indications : Axo Flow is indicated for direct filling of class V (Cervical Caries), root erosions, wedge shaped defects. Fillings of anterior teeth (Class III & IV). Small fillings in posterior teeth. Repair of composite and ceramic veneers Block out of undercuts. Adhesive cementing of composite and ceramic restorations.

Instructions for use : Apply the material with applicator tip directly in the cavity and pre contour with a suitable instrument. The layer should not be thicker than 2mm. Light cure for 40 sec using standard curing light.

Warning : Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally. Contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician.

Contraindications : It is contraindicated for use with patients who have a history of severe allergic reaction to methacrylate resins.

Precautions : Eugenol containing materials should not be used with this product because they may effect polymerization and interfere with the hardening process. Being sensitive to heat and light, the syringes should be kept away from direct sun light.

Storage Conditions :

Store at temperature between 10°C to 24°C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf Life : 3 years from the date of manufacturing.

Presentation :

1 x 2g Syringes, 3 x applicator Tips

Shades : A1, A2 & A3

Disposal Conditions :


To be disposed off based on the prevailing rules of the country in which it is used.



Manufactured by
Dentaxon
3/222C, 3/222D
Malappuram, Kerala -679328
info@dentaxon.com
www.dentaxon.com

CE
0123

Revision :	
Date :	

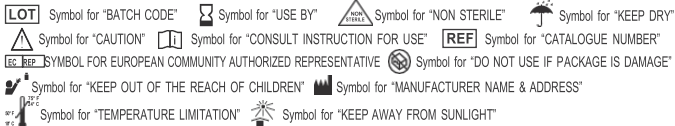

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.