

AXO CURE

DRY SOCKET PASTE



Composition:

Butamben, Iodoform, Eugenol, excipients

Properties:

Nonresorbable temporary Socket dressing paste.

Axo Cure is a fiber paste with specific odor used for antiseptic treatment of extraction socket

Axo Cure contains .Axo Cure contains:

- 1) Butamben is a local anesthetic, acts by inhibiting the voltage-gated calcium channels in dorsal root ganglion neurons.
- 2) Iodoform – antiseptic, possesses durable antibacterial effect
- 3) Eugenol, enhances antiseptic effect and thanks to its analgesic benefit, relieves pain
- 4) Excipients, provide sustained therapeutic effect.

Axo Cure is not resorbable. Paste washes out of socket over time, which helps in wound healing. During follow-up there's no need to extract paste remains, i.e. no soft tissue traumatizing.

Dental product is produced for using in aggressive biological medium with the temperature range between 32°C and 42°C (in oral cavity at the temperature (37±1)°C and humidity 60-90 %).

Indications:

Axo Cure is used during:

- Socket treatment after tooth extraction
- Alveolgia prophylaxis
- Acute alveolitis treatment

Contraindications:

Iodine or its derivatives allergy

Do not use for patients with blood-clotting disorder

Do not swallow

Side Effects:

Can be the reason for skin irritation when in contact. Use gloves during work.

As far as all terms of storing, transporting and application are observed, there are no other side effects

Method Of Use:

Lay on a bandage roll small amount of paste and place it in socket for 2-6 days, or, make a small pellet of paste and place it in socket.

Do not suture, paste is not resorbable.

The medication works for several days. That is important to give patient a warning not to swallow the paste and not to rinse mouth hard.

Presentation:

Paste (jar) 15g

Storage:

Store at the temperature from 5°C up to 25°C

Tightly close the jar after use

Do not use after expiry date

Shelf life – 3 years

Not For Medicinal use, For use in Dentistry By Dental Professional only




Manufactured by
Dentaxon
3/222C, 3/222D
Malappuram, Kerala -679328
info@dentaxon.com
www.dentaxon.com

Revision :	
Date :	

 Symbol for "HARMFUL IRRITANT"

 Symbol for "CORROSIVE"

 Symbol for "BATCH CODE"  Symbol for "USE BY"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"

 Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE"  Symbol for "CATALOGUE NUMBER"

 SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE  Symbol for "DO NOT USE IF PACKAGE IS DAMAGED"

 Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER NAME & ADDRESS"

 Symbol for "TEMPERATURE LIMITATION"  Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.