

AXO TEMP

EUGENOL FREE TEMPORARY FILLING



Discription:

Axo Temp is a paste consistency, adaptive in nature temporary filling material which hardens upon contact of moisture in Oral Cavity. Marginal seal of material is excellent , so it makes tight seals and closure of working area.

Composition:

Zinc Oxide, Calcium Sulphate, Poly Vinyl Acetate Resin, Flavours and additives.

Properties and Method of Application:

Axo Temp is inserted with small portion with any suitable filling instrument in to cavity, due to its excellent mould ability it spreads in to cavity with gentle pressure to adapt material with cavity wall. We recommended in to wet cavity for better adaptation. Remove excess material by wiping it off with a moist cotton pellet towards margin of the cavity. It hardens under moisture, removal of material is so ease, you can remove material with use of diamond rotary burs or sharp filling instrument.

Indication:

As a temporary filling material for Root Canal Procedure and Cavity Restoration and Inlays and implant abutment sealer .

Contraindication:

Drug idiosyncrasy

Use with carefully if person is allergic to PVC resin

Do not use directly on exposed Pulp area as a temporary or permanent closure

Side effects:

No any known side effects as far as all terms of application, transportation and storage are observed

Presentation:

Axo Temp 40g jar

Storage:

Store at 5 °C to 25 °C, in a well sealed container Attention! Close material tightly after use

The material must not be used after the expiry date


Shelf life – 3 years

Not For Medicinal use, For use in Dentistry By Dental Professional only

Manufactured by
Dentaxon
3/222C, 3/222D
Malappuram, Kerala -679328
info@dentaxon.com
www.dentaxon.com

Revision :	
Date :	

 Symbol for "HARMFUL IRRITANT"

 Symbol for "CORROSIVE"

 Symbol for "BATCH CODE"  Symbol for "USE BY"  Symbol for "NON STERILE"  Symbol for "KEEP DRY"

 Symbol for "CAUTION"  Symbol for "CONSULT INSTRUCTION FOR USE"  Symbol for "CATALOGUE NUMBER"

 SYMBOL FOR EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"

 Symbol for "KEEP OUT OF THE REACH OF CHILDREN"  Symbol for "MANUFACTURER NAME & ADDRESS"

 Symbol for "TEMPERATURE LIMITATION"  Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.